



Orange Aid[®]

Pressure management gel products



Reduce The Risk Of Pressure & Positioning Injuries



Orange Aid® Gel Products improve patient positioning by distributing pressure and heat during surgical procedures. Orange Aid products are specifically designed to stand up to the rigors of the OR environment and can be used with all surgical tables. These unique pressure management gel pads are made without natural rubber latex and the distinctive orange color helps avoid inadvertent disposal.

Prior to positioning Orange Aid Gel Products clinical assessment guidelines and objectives should be used to identify each patient's risk for potential pressure-related problems associated with lengthy surgeries.



Orange Aid® Features

- Durable and strong
- Non-absorbent
- Evenly distributes weight
- Not made with natural rubber latex
- Will not bottom out
- Easy to clean
- Reusable



Cleaning. Care. Storage.

The integrity and shelf life of Orange Aid products can be increased significantly by following simple protocols for cleaning, care and storage.

- Disinfect Orange Aid product between patient uses with standard operating room cleaning solutions that do not contain alcohol
- Rinse Orange Aid with water and wipe dry after cleaning
- Do not allow gel product to soak in cleaning fluids for prolonged periods of time
- Do not autoclave, freeze, or microwave
- Do not puncture, cut or tear Orange Aid products
- Keep Velcro® attachments free of foreign matter to ensure adequate closure
- Warm Orange Aid products by placing in a blanket warmer or by placing below a patient warmer. Do not exceed temperatures of 100°F (37°C)
- Store Orange Aid products flat when possible to decrease the potential for damage; large pads may be folded loosely to avoid creasing
- Do not store heavy or sharp objects on top of Orange Aid products

Key Products

- General Table Pads
- Head Rings
- Arm Board Pads
- Ulnar/Brachial Protectors
- Ankle Stirrup Straps
- Heel Elevator Cups
- Boot / Stirrup Liners
- Heel Protectors
- Heel and Elbow Protectors
- Knee Crutch Pads
- Universal Patient Positioners
- Patient Safety Strap with Gel Pad
- Four Poster Frame Covers
- Wilson Frame Pad
- Universal Perineal Post Pad
- Universal Traction Boot Liner
- Lateral Comfort Positioner
- Bean Bag Cover
- Repair Kit
- Dome Positioner
- Chest Rolls



Orange Aid® Products



Table Pads

Enhances pressure distribution on selected sections or full length of any OR table.

REF

10101	Table Pad, Hip or Foot Section	20.0" x 20.0" x 0.5" (51 x 51 x 1.3 cm)
10102	3/4 length OR Table Pad	46.0" x 20.0" x 0.5" (117 x 51 x 1.3 cm)
10103	Full length OR Table Pad	72.0" x 20.0" x 0.5" (183 x 51 x 1.3 cm)
10104	Table Pad, Head Section	9.0" x 20.0" x 0.5" (23 x 51 x 1.3 cm)
10105	Table Pad, Torso Section w/ Perineal Cutout	39.5" x 20.0" x 0.5" (100 x 51 x 1.3 cm)
10106	Table Pad, Foot Section w/ Perineal Cutout	20.0" x 20.0" x 0.5" (51 x 51 x 1.3 cm)
10107	Table Pad Three Piece Set	Includes: 10104, 10105, and 10106
10108	Table Pad Three Piece Set	Includes: 10104, 10105, and 10101



Head Rings

Gently supports the head while protecting the face, neck and ears.

Closed Head Rings

10201	Neonatal	3.5" x 1.5" x 0.75" (9 x 4 x 2 cm)
10202	Adolescent	5.5" x 2.25" x 1.25" (14 x 6 x 3 cm)
10203	Adult	8.0" x 3.0" x 1.75" (20 x 8 x 4.5 cm)

Open Head Rings

10205	Neonatal	3.5" x 1.5" x 0.75" (9 x 4 x 2 cm)
10206	Adolescent	5.5" x 2.5" x 1.25" (14 x 6 x 3 cm)
10207	Adult	8.0" x 3.0" x 1.75" (20 x 8 x 4.5 cm)



Arm Board Pads

Helps reduce pressure on ulnar, radial and median nerves.

10301	Standard	15.0" x 5.5" x 0.38" (38 x 14 x 1 cm)
10303	Long	24.0" x 4.5" x 0.38" (61 x 11 x 1 cm)
10302	Long-Extra Wide	23.0" x 7.5" x 0.38" (58 x 19 x 1 cm)



Ulnar/Brachial Nerve Protectors

Wraps sensitive nerve locations with an extra layer of protection.

10305	Oval	13.0" x 5.0" x 0.38" (33 x 12.5 x 1 cm)
10306	Rectangular	17.0" x 11.0" x 0.38" (43 x 28 x 1 cm)



Chest Rolls

Replaces rolled towel bolsters to promote stability and ventilation regardless of patient weight.

10811	Flat Bottom	12.0" x 4.0" x 3.0" (30 x 10 x 8 cm)
10812	Flat Bottom	16.0" x 4.0" x 3.0" (41 x 10 x 8 cm)
10813	Flat Bottom	20.0" x 4.0" x 3.0" (51 x 10 x 8 cm)
10814	Flat Bottom w/ Ultra Lite Core	12.0" x 6.0" x 4.0" (30 x 15 x 10 cm)
10815	Flat Bottom w/ Ultra Lite Core	16.0" x 6.0" x 4.0" (41 x 15 x 10 cm)
10816	Flat Bottom w/ Ultra Lite Core	20.0" x 6.0" x 4.0" (51 x 15 x 10 cm)

Orange Aid® Products



Candy Cane Covers

Added protection on lithotomy leg holders, particularly with electrocautery devices.

REF

10400	Set of 2	24.0" x 2.0" (61 x 5 cm)
-------	----------	--------------------------



Ankle Stirrup Straps

Permits padding and full view of feet in lithotomy position.

10401	Set of 2	12.25" x 2.0" x 0.38" (31 x 5 x 1 cm)
10411	Set of 2, Large	17.5" x 2.0" x 0.38" (44 x 5 x 1 cm)



Heel Elevator Cups

Supports foot weight and adds pressure protection during long surgical procedures.

10402	Set of 2	7.25" x 4.0" x 2.75" (18 x 10 x 7 cm)
-------	----------	---------------------------------------



Boot/Stirrup Liners

Fits all Lithotomy Boots/Stirrups to protect lower legs and heels.

10403	Set of 2	24.0" x 9.0" x 5.5" (61 x 23 x 14 cm)
-------	----------	---------------------------------------



Heel Protectors

Helps alleviate heel pressure during long procedures.

10404	Set of 2	Universal Size
-------	----------	----------------



Knee Crutch Pads

Comfort and protection; ideal for patients using standard leg supports.

10502	Set of 2, Small	9.0" x 11.0" x 0.38" (23 x 28 x 1 cm)
10503	Set of 2, Medium	11.0" x 17.0" x 0.38" (28 x 43 x 1 cm)
10505	Set of 2, Large	20.0" x 20.0" x 0.38" (51 x 51 x 1 cm)



Universal Patient Positioners

Safe and effective procedure positioning; ideal for knees, heels, shoulder, lateral and axillary roll support.

10600	Small, Long	13.0" x 3.0" x 2.5" (33 x 8 x 6 cm)
10601	Small, Regular	6.0" x 3.0" x 2.5" (15 x 8 x 6 cm)
10602	Medium	10.0" x 4.5" x 2.5" (25 x 11 x 6 cm)
10603	Large	14.0" x 4.5" x 2.5" (36 x 11 x 6 cm)



Dome Positioner

Low profile flat-bottomed positioner; ideal for knees, feet, rib cage, and axillary patient positioning.

10620	Universal Dome Positioner	14.0" x 6.0" x 3.0" (36 x 15 x 8 cm)
-------	---------------------------	--------------------------------------



For product information or to order contact your account manager or call 1-800-777-4674.



Patient Safety Strap With Gel Pad

Provides stability and tissue protection on any operating table.

REF

10604	Patient Safety Strap with Gel Pad	Includes a 96 in. (244 cm) polyester web strap with plastic buckle and gel pad
10920	Gel Pad only	20.25" X 3.25" X 0.38" (51 x 8.25 x 1.0 cm)



Four Poster Frame Pad Covers

Provides increased protection while patient is in the prone position.

10701	Set of 4	6.75" x 6.75" (17 x 17 cm)
-------	----------	----------------------------



Wilson Frame Pads

Covers any Wilson Frame. Adds patient protection during spinal procedures.

10702	Set of 2	29.0" x 10.0" x 1.0" (74 x 25 x 2.5 cm)
10703	Set of 2	29.0" x 10.0" x 0.5" (74 x 25 x 1.3 cm)



Universal Perineal Post Pad

Provides extra protection during orthopedic fracture cases involving mechanical traction.

10704	Universal Perineal Post Pad	11.0" x 9.0" x 0.38" (28 x 23 x 1 cm)
-------	-----------------------------	---------------------------------------



Universal Traction Boot Liner

Adds protection. Slips easily into any traction boot.

10706	Universal Traction Boot liner	Universal Size
-------	-------------------------------	----------------



Bean Bag Cover

Evenly distributes the pressure relieving properties of gel while patients are in the lateral position.

10708	Universal Bean Bag Cover	38.0" x 23.0" x 0.5" (97 x 58 x 1.3 cm)
-------	--------------------------	---



Lateral Comfort Positioner

Supports patients comfortably while in the lateral position. Includes a relief channel for the lower arm.

10715	Lateral Comfort Positioner with Gel Pad	
-------	---	--

Repair Kit

Provides temporary repairs to torn pads without the use of tape.

10910	Orange Aid Repair Kit	
-------	-----------------------	--


mizuhosi.com



30031 Ahern Avenue
Union City, CA 94587-1234 USA
Telephone: 510-429-1500
Toll Free: 800-777-4674
Fax: 510-429-8500
Outside USA: +1-510-429-1500
mizuhosi.com



Emergo Europe
Prinsessegracht 20
2514 AP The Hague
The Netherlands

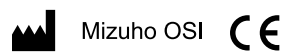

Note: Mizuho OSI is constantly improving its products. All specifications are subject to change without notice. All dimensions listed are approximate. Orange AID® is a registered trademark of Mizuho OSI • Mizuho OSI is a Delaware Corporation. Manufactured in the USA.

2016 ©Mizuho OSI

REF NW0662 REV D